800 mg

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When to use Nasaleze

Take **Nasaleze** before symptoms start.

Nasaleze is a preventative treatment in the fight against cold viruses.

Nasaleze has been clinically proven to help stop the onset of a cold or shorten its duration.

Nasaleze is clinically proven to work effectively in adults and children from 3 years.

If symptoms of cold have already started before taking **Nasaleze**, you can still apply **Nasaleze** to shorten its duration.

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Nasaleze can be used in addition to medicines, as part of a combination therapy strategy in the fight against cold.

How does Nasalezework?

The most common route of infection for colds is through the nose. Cold viruses enter the nose and attach to the nasal lining, where they multiply and spread. This then leads to symptoms, such as a blocked or runny nose, sore throat, headache and coughing. In the initial stages of a disease you often cannot distinguish between the common cold. Influenza is a potentially serious disease; the definitive diagnosis requires a physician.

Nasaleze is a powder nasal spray. The powder turns to a gel in the nose which creates a barrier to shield, protect and deactivate airborne germs and viruses.

Nasaleze can shorten the duration and severity of cold like symptoms.

Nasaleze can help stop the reappearance of virus-related cold like symptoms.

The **Nasaleze** treatment action has been proven in 4 clinical studies.

SPRAY → TRAP & DEACTIVATE → HELPS DEFEND AGAINST COLD

Who can use Nasaleze?

Nasaleze is suitable for adults, athletes, pregnant and breast-feeding women and children from 3 years.

Keep out of reach of young children.

Nasaleze does not contain any drugs or medicines.

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If Nasaleze is not adequately relieving your symptoms...

Consult your physician about what medication you could take to supplement your **Nasaleze** in the fight against cold.

Read the above guidance carefully and try **Nasaleze** using the 5 steps.

How to store Nasaleze

Do not rinse the bottle with water or other liquid as this may block the bottle.

Do not use if tamper evident seal is broken.

Always replace the cap.

Store at room temperature, or between 5°C and 30°C

Once opened use within 6 months.

Do not refrigerate or freeze.

Contraindications

Not suitable for children under 3 years.

Do not use this product if you are sensitive to cellulose, mint or garlic.

Avoid contact with eyes.

Avoid contact with open wounds.

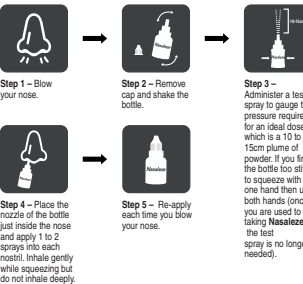
If powder gets into your eye, rinse with water.

To avoid contamination do not use this product for more than one person.

Very few cases of allergic reactions to this product have been reported. Minor reports received have been blocked nose, runny nose, sneezing, sore throat. If you experience reactions more severe than this then use of the product should be discontinued immediately and if symptoms persist, consult your physician.

How to use Nasaleze

To defend against cold, **Nasaleze** needs to be applied 2 to 3 times per day to maintain an effective barrier. Or if symptoms have already started take **Nasaleze** 3 times per day until symptoms disappear.

During first use, you may experience a tingling or burning sensation in the nose or light sneezing. This is a transient sensation that will go away as you become used to taking **Nasaleze**.

You may also feel some form of irritation, especially if your nasal membranes are already inflamed or irritated. This will subside quickly.

Some individuals can feel a slight sensation of a blocked nose, this is normal and is the feel of the barrier created by **Nasaleze**.

If you experience serious discomfort after application of **Nasaleze** consult your physician.

If concurrent use of another nasal spray is required, **Nasaleze** should be re-applied afterwards so that the barrier created is not disturbed.

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Nasaleze Ltd, Unit 3, Woodside House,
Imported by: TRUSTPHARM Ltd,